
Technowood®

SSS Guide Wire BKM Shape, 1.5 mm unique and effective

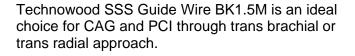






The distal shape possesses a unique curve and is a slightly opened J curve, and makes it much easier to advance into catheter hub without the use of tedious guide wire tip straightner.

A second angled curve provides a featured selective control of the guide wire to facilitate the guide wire advancement when challenged by the tortuous subclavian artery.

Guide wire body is stiff enough to straighten up the distal shape of the PCI guiding catheter while the distal tip of the guide wire is soft and safe toward the blood vessel walls.

Technowood SSS Guide Wire BK1.5M is uniquely engineered with modified distal J curve and the secondary angled curve; each curve designed for specific clinical needs.

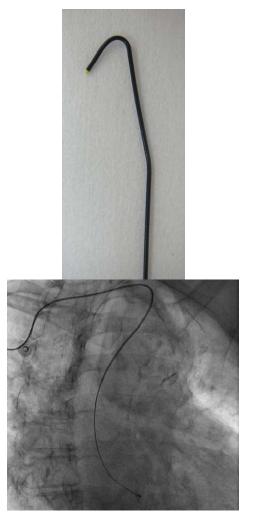

***JAPAN**

Technowood International Corporation 30-19 Kohoku, 4-Chome Adachi-Ku, Tokyo 123-0872 Japan

Tel	81-3-3898-5252
Fax	81-3-5898-5252
Web Site	www.technowood.co.jp
Contact	info-tic@technowood.co.jp

★USA Technowood America Corporation 3001 Red Hill Avenue Building 2, Suite 201 Costa Mesa, CA 92626 U.S.A. Tel 1-714-434-8713 Fax 1-714-434-8715 Web Site www.technowood.com Contact info@technowood.com

***Distributed by:**

Tel Fax Web Site Contact

SSS Guidewire Clinical Update 05