
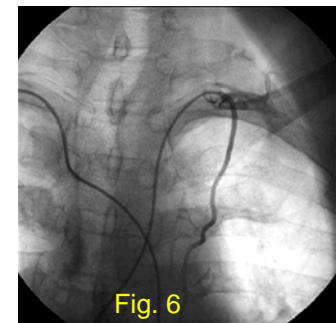
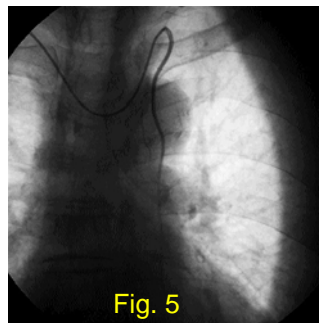
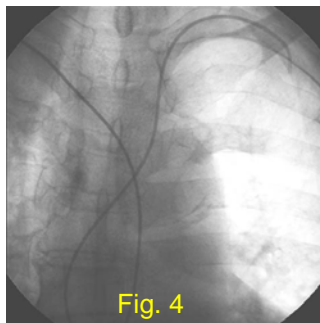
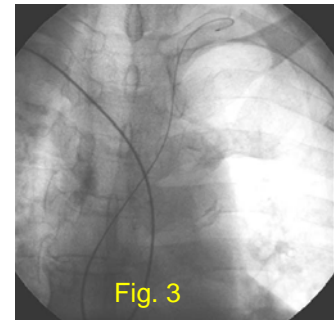
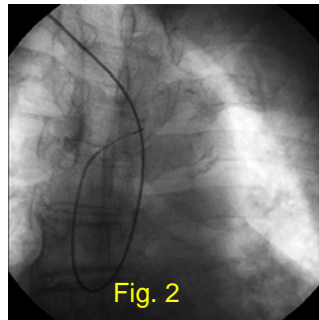
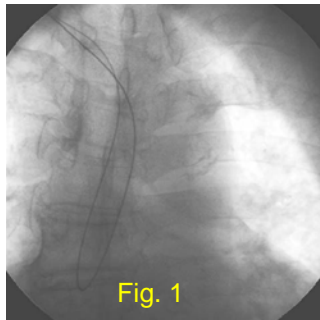


# Technowood® SSS Guide Wire

Left internal mammary artery angiogram through right radial artery approach with Technowood SSS Guide Wire, 3 mm "J" 150 cm - JR 3.5 Catheter

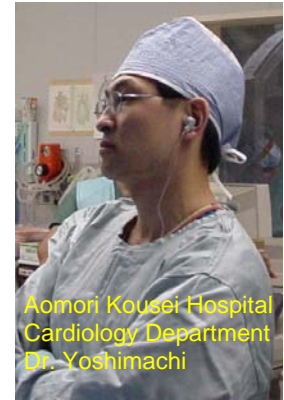

**Background:** Left internal mammary artery angiogram is often approached by right radial artery entry. Our preferred approach is to use the standard Judkins catheter for right coronary artery (JR 3.5) together with Technowood SSS Guide Wire.

**Methods:** Technowood SSS Guide Wire is positioned and rotated counterclockwise inside aorta and advanced to right brachial artery (Fig. 1). Catheter is positioned toward descending aorta, and withdraw Guide Wire halfway (Fig. 2). Advance the Guide Wire toward left subclavian artery (Fig. 3). Advance Guide Wire as far as possible and the catheter to follow over the Guide Wire (Fig. 4). Remove Guide Wire and adjust the catheter positioning for left internal mammary artery angiogram (Fig. 5). Otherwise, Guide Wire is remained inside the catheter and adjust the catheter positioning and remove Guide Wire and then left internal mammary artery angiogram is performed (Fig. 6).

**Conclusions:** Left internal mammary artery angiogram through right radial artery approach can be safely performed using Technowood SSS Guide Wire and Judkins catheter.

## ★ JAPAN

**Technowood International Corporation**  
30-19 Kohoku, 4-Chome  
Adachi-Ku, Tokyo 123-0872  
Japan

Tel: 81-3-3898-5252  
Fax: 81-3-3898-5252  
Web-Site: <http://www.technowood.co.jp>  
Contact: [info-tic@technowood.co.jp](mailto:info-tic@technowood.co.jp)

## ★ USA

**Technowood America Corporation**  
3001 Red Hill Avenue  
Building 2, Suite 201  
Costa Mesa, CA 92626  
U.S.A.

Tel: 1-714-434-8713  
Fax: 1-714-434-8715  
Web-Site: <http://www.technowood.com>  
Contact: [info@technowood.com](mailto:info@technowood.com)

## ★ Philippines

**Technowood Manufacturing Corporation**  
N2840 Bayanihan Street Extension  
CSEZ Clarkfield, Pampanga 2009  
Philippines

Tel: 63-45-599-5570 or 63-45-599-5571  
Fax: 63-45-599-5574  
Web-Site: <http://www.technowood.com.ph>  
Contact: [info@technowood.com.ph](mailto:info@technowood.com.ph)